


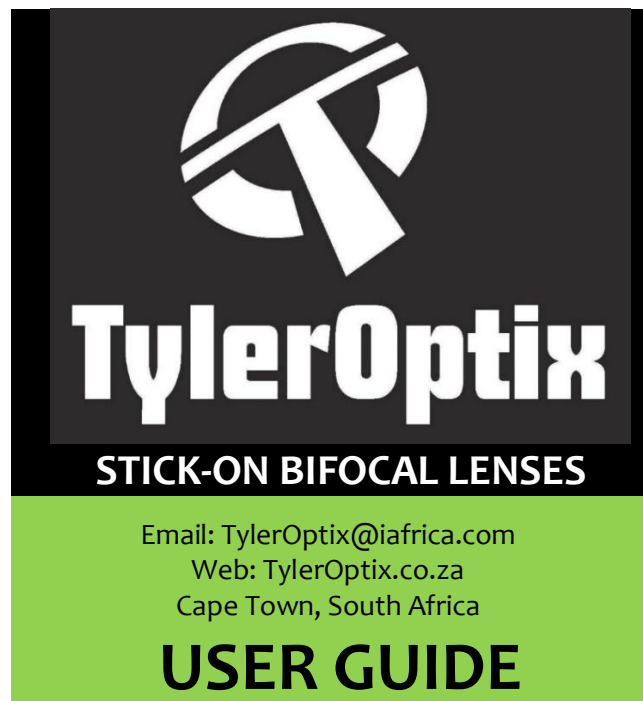

INSTALLATION & PREPERATION OF STICK-ON LENSES



Reusable

1. Carefully remove the plastic lenses from the container.
2. Wash lenses, eyewear and hands with a oil free liquid soap such as dishwashing liquid and rinse the soap off with clean warm water.
3. Leave a drop of water in the well of the eyewear.
4. Place one lens with the curved side onto the matching inside surface of the eyewear on the water.
5. Position the lens as close as possible to the nose bridge side and bottom of the eyewear.

6. Press gently on the lens to remove the air bubbles underneath the lens starting from one side to the other.
Please do not press too hard to squeeze out all the water as this helps create the bond

7. Wear eyewear to check proper positioning. At this point the lenses can be moved and adjusted until correct. Make sure the lenses do not block distance viewing. Repeat steps 1. to 6. for the second lens.
8. Use a lint-free cloth or tissue to carefully pat away any excess water from your lenses. Allow the lens to dry in place for at least 24 hours or more. Note that the view through lens might be a little blurry until completely dry.

NOTE:

If dry patches appear under the lens, it means it has not bonded properly and needs to be removed and reapplied repeating the installation steps. If not reapplied, the dry air pocket could spread causing the lens to unexpectedly fall off the eyewear.

PRECAUTIONS & OTHER USES FOR STICK-ON LENSES

The lenses can be simply removed by wetting and placing a fingernail under a lens corner and carefully lifting and removing it. Store it in the container it came in when not in use. Both lenses are the same and there is no left or right or back or front and can be fitted anyway.

Customising

The lenses will fit most eyewear but can be customised and trimmed to a smaller size with a very sharp scissors. Reinstall as per the steps in the installation USER GUIDE.

Bifocal for Top Instrument Reading

A second pair of lenses can be fitted to the upper outer corners.

Bifocal For Golfers

They allow reading without obscuring distance or lower vision.
Only fit one lens onto the upper outer corner: right side segment for right-hand golfers and left lens for left-handed golfers.

Bifocal For Divers

Lenses can be applied to the inside of diving masks for dial reading and close up underwater viewing. Please rinse the salt off the mask with fresh water before fitting. Once lenses have been fitted allow to dry for 48 hours before diving.
Once dry they will not come off even if the mask becomes flooded.

Cleaning

Lenses can be washed with a oil free liquid soap such as dishwashing liquid. Rinse the soap off with clean water.

NOTE: Do not clean with solvents or spirits as this will destroy the material and lens clarity.